

User Manual



SPINAL REHAB SOLUTIONS, INC.[®]
"IMPROVING QUALITY OF LIFE"



SR 100 AMBULATORY CERVICAL SPINAL-AIR TRACTION COLLAR

RECOMMENDED FOR:

- Cervical Pain Syndrome • Pinched Nerve • Cervical Radiculopathy • Herniated Disc
- Bulging Disc • Degenerative Disc Disease • Chronic Neck Pain • Chronic Migraine




Each Unit Includes:

- 1 Chin Collar
- 2 Cervical Traction Air Bladder
- 3 Air Pump With Gauge
- 4 Extension Hose
- 5 User Manual
- 6 Laundry Bag



Hand Air Pump

Terms of Each Part of SR 100 For Neck (Model: SR 100)

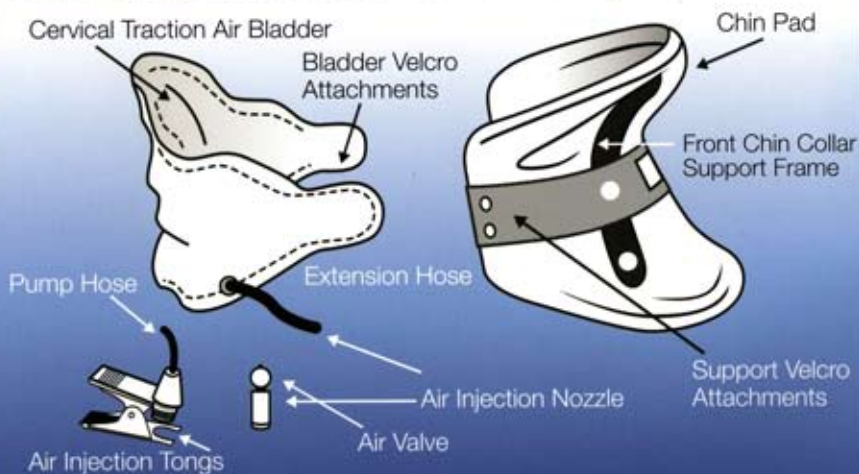

Precautions:

- 1 Only use hand air pump that is provided with device. The use of a different inflation pump could damage device valve and void warranty.
- 2 While inflating device, never exceed more than 14 psi air pressure allowing pressure gauge needle to enter into the red area of hand air pump pressure gauge.
- 3 Patients with heart disease and or hypertension should consult a physician prior to using the SR 100 Ambulatory Cervical Spinal-Air Traction Collar.
- 4 Prior to washing the SR 100 Ambulatory Cervical Spinal-Air Traction Collar Bladder make sure the air valve is closed completely by turning air valve clockwise until firmly tightened to avoid water entering into the device.
- 5 Handle with care; Dropping hand air pump can result in malfunction. Keep sharp objects away from Collar Air Bladder.



Wearing Instructions:

1. Before use, make sure that the traction air bladder of this device is completely deflated by turning air valve counterclockwise to release air.
2. After the traction air bladder is completely deflated, return valve to locked position by turning valve clockwise until the valve is fully tightened.
3. Position the front support frame by placing your chin within the designated chin pad. Fasten the traction air bladder to the chin collar with the Velcro fasteners making sure that both the traction air bladder and chin collar are centered (the use of a mirror while positioning this device on your body is recommended) around your neck. The device should be fastened around your neck fairly snug, however not too tight.
4. Connect the hand air pump valve to the device air valve by opening the spring loaded clamp by pushing the clamp arms together, insert the tongs of the hand air pump valve into the designated groove of the device air valve and securely place hand pump air valve over the device air valve.
5. Proceed to inflate device by initiating hand air pump action. Continue to inflate device until the needle in the hand pump air pressure gauge moves into the green / orange area (10-13 psi). Caution: Do not allow needle to exceed into the red area of the hand pump air pressure gauge. By doing so can damage the air bladder.
6. Remove the hand air pump valve and the SR100 Ambulatory Cervical Spinal-Air Traction Collar is ready to wear.
7. For more comfort as you are wearing the device, you may at anytime release some air pressure out of the device by pushing down briefly on the air valve stem.
8. Recommended treatment wear time is 36-60 minutes.
9. Prior to removing the device from your body, you MUST first release the air from the air traction bladder by turning the air valve counterclockwise to release air.

Optimal Wearing and Maintenance Recommendations:

WEARING INSTRUCTIONS

RECOMMENDED FOR:

- Cervical Pain Syndrome • Pinched Nerve
- Cervical Radiculopathy • Herniated Disc
- Bulging Disc • Degenerative Disc Disease
- Chronic Neck Pain • Chronic Migraine

NOT RECOMMENDED FOR

1. High Blood Pressure.
2. Cerebrovascular Disease
3. Cervical Tumor
4. Inflammatory Disease of Cervical Vertebrae
5. Severe Cervical Spine Instability
6. Severe Osteoporosis
7. Post Spine Surgery (90 days)

WASHING INSTRUCTIONS

1. Prior to washing the SR100 Ambulatory Cervical Spinal-Air Traction Collar Bladder, make sure that the air valve is closed completely by turning valve stem in the clockwise direction until firmly tightened to avoid water entering into the traction air bladder.
2. Place the device in the mesh laundry bag (included) to prevent damage.
3. Place the device in the washing machine, program the washing machine for a gentle cold wash cycle. A gentle, delicate detergent is recommended.
4. After washing, remove device from mesh laundry bag, allow to air dry (*keep away from direct sunlight*).
5. Front Support Frame Chin Collar can only be hand washed.



**The SR 100 Ambulatory Cervical Spinal-Air Traction Collar
is manufactured under the following standards:**

ISO 9001:2000 (International Quality Management System)

ISO 13485 (International Quality System Medical Devices)

EN 46002 (European Quality System Medical Devices)

CE Mark

FDA ER No. 9611754 and Device Listing No. B 115761 in U.S.A.

Notification No. BE/CA01 /2-03702-000011 Belgium

Notification No. BE/CA01 /2-03702-00002 Belgium

Clinical consultation:

Department of Orthopedic Surgery,
Inje University, Seoul Paik Hospital, Korea

User Manual 6 - Month Warranty Limited Warranty:

SR 100 Ambulatory Cervical Spinal-Air Traction Collar is warranted against defects in material and workmanship. To obtain service under this warranty, please mail warranty card at time of purchase.

Spinal Rehab Solutions, Inc. warrants its products to be free from defects in materials and workmanship for six months from the date of purchase, except as provided below under the limited warranty.

The exceptions or exclusions to this warranty are as follows:

1. Any warranty claim that is submitted after the limited warranty period, or without proper proof of purchase will not include the cost of part, labor, or delivery.
2. This warranty shall not apply to products which have been subjected to misuse, neglect, alteration, modification, or attachments, either caused by the original purchaser, shipping, storage, accident, fire, flood, or acts of God.

In the event of a defect in material or workmanship covered by this warranty, Spinal Rehab Solutions, Inc. will repair or replace, at their discretion, without charge the warranted device within a reasonable amount of time. We reserve the right to require damaged device to be returned to us. Natural markings such as scars, brands, grain variations, wrinkles, color variations, etc. will be considered normal characteristics and not construed as defects. We cannot warrant material against scratching or scuffing, as all material is subject to blemishes with use.

For claims or questions concerning this warranty, please contact Spinal Rehab Solutions, Inc. Customer Service department by phone 1-702-586-5107, or e-mail info@spinalrehabolutions.com

Be prepared to provide your name, address, daytime phone number, model, date of purchase, and the facility where it was purchased. Spinal Rehab Solutions, Inc. requires that the original sales receipt be submitted with all requests to confirm that you are the original purchaser.

This warranty gives you specific rights, and you may also have other rights, which vary from state to state.

All warranties are limited to the original purchaser for normal use.

Product Registration / Warranty

First Name: _____ MI: _____ Last: _____

Address: _____

City: _____ State: _____ Zip: _____

Phone: _____ Evening: _____

Product Information:

Date of Purchase: _____

Purchase Price: _____

Dr./Practice Name: _____

How did you become aware of Spinal Rehab Solutions, Inc.?

Mail to:

Spinal Rehab Solutions, Inc.
177 Cassia Way Ste B111
Henderson, NV. 89014-6646
Direct: 702-586-5107

